

PRODUCT CATALOG

Medicinal Maggots™

Medicinal Maggots™

Disinfected *Phaenicia* (= *Lucilia*) *sericata* larvae

Supplied as **maggot-impregnated gauze**, 2 gauze pads per vial. Each vial contains about 250 - 500 larvae (approximately 200 larvae per gauze pad).

Catalog Number: MM250 Call for Price



Maggot MegaPak

Items Included:

Maggot MegaPak offers customized dressing options and everything you need, at one price. For simplicity, we have prepared the following complete treatment package for you, excluding shipping!

Medicinal Maggots™ (1 vial)
LeGlu (if needed)
Hydrocolloid Pad (1 pad)
Skin protectant (2)
Tape (1 roll)
Gauze (1 pack)


Dressing of your choice

- LeSoc™
- Sheer Comfort™

SHIPPING NOT INCLUDED.

Catalog Number: MegaPk Call for Price

Maggot Dressings, à la carte

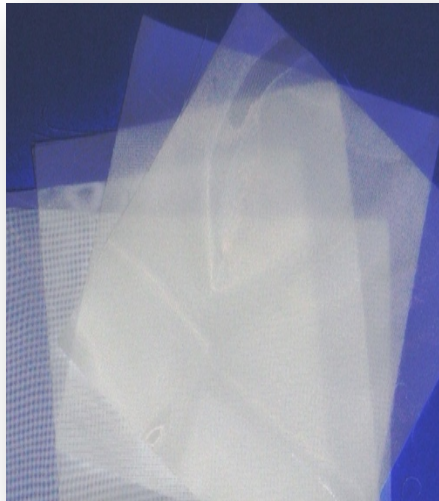

Sheer Comfort™, sterile

Polyester net dressing to keep the maggots on the wound. Using adhesive glue and tape, bond to a hydrocolloid pad or other foundation on the skin to build a maggot cage around the wound.

<u>Size:</u>	<u>Catalog Number:</u>	<u>Price:</u>
4 inches x 4 inches, sterile	CC44	Call
8 inches x 8 inches, sterile	CC88	Call
12 inches x 12 inches, sterile	CC1212	Call
24 inches x 24 inches, sterile	CC2424	Call
Nylon stocking, sterile knee-high	CCKH	Call
Nylon pantyhose, sterile	CCPH	Call

Maggot Dressings, à la carte



LeSoc™, sterile

Stocking-like dressing constructed from fixed-weave polyester monofilament netted fabric (like our Sheer Comfort™). Use to dress wounds with complicated 3-dimensional surfaces, like toes, feet, stumps. Similar to the Sterile Nylon Stockings. This fixed-weave fabric better prevents against escaping maggots. Not for areas that may swell or become constricted.

<u>Size:</u>	<u>Catalog Number:</u>	<u>Price:</u>
3 inches x 4 inches LeSoc™ - Two-Finger sized, sterile Length: 4" / 10 cm; Width: 3" / 7.5 cm; Circumference: 6" / 15 cm; Diameter: 1.9" / 4.5 cm	LS3X4	Call
4 inches x 6 inches LeSoc™ - Four-Finger-sized, sterile Length: 6" / 15 cm; Width: 4" / 10 cm; Circumference: 8" / 20 cm; Diameter: 2.5" / 6.4 cm	LS4X6	Call
6 inches x 8 inches LeSoc™ - Hand-sized, sterile Length: 8" / 20 cm; Width: 6" / 15 cm; Circumference: 12" / 30 cm; Diameter: 3.8" / 9.5 cm	LS6X8	Call
8 inches x 12 inches LeSoc™ - Anterior Foot, sterile Length: 12" / 30 cm; Width: 8" / 20 cm; Circumference: 16" / 40 cm; Diameter: 5" / 12.5 cm	LS8X12	Call
12 inches x 18 inches LeSoc™ - Boot-sized, sterile Length: 18" / 45 cm; Width: 12" / 30 cm; Circumference: 24" / 60 cm; Diameter: 7.5" / 19 cm	LS12X18	Call
18 inches x 24 inches LeSoc™ - Leg-sized, sterile Length: 24" / 60 cm; Width: 18" / 45 cm; Circumference: 36" / 91 cm; Diameter: 11.4" / 29 cm	LS18X24	Call
24 inches x 36 inches LeSoc™ - Thigh-high, sterile Length: 36" / 91 cm; Width: 24" / 60 cm; Circumference: 48" / 122 cm; Diameter: 15" / 38 cm	LS24X36	Call

Warning — since LeSoc™ is not elastic, care must be taken to ensure that it does not constrict or prevent adequate blood flow.

Maggot Dressings, à la carte



Hydrocolloid Pad, sterile

Used to make custom maggot therapy dressings at the bedside. Simply cut a hole in the pad to match the wound shape, place remaining pad as a foundation around the wound, place the maggots on the wound bed, and glue your porous cover (i.e., Sheer Comfort™ polyester net) to the hydrocolloid foundation.

Catalog Number: Price:

Hydrocolloid pad, 4 inches x 4 inches	HC44	Call
Hydrocolloid pad, 6 inches x 6 inches	HC66	Call
Hydrocolloid pad, 8 inches x 8 inches	HC88	Call

(Brands may vary)



LeGlu Medical Adhesive By Nu-Hope

For gluing polyester net to hydrocolloid pad or other foundation. Shake (the bottle) well just before use.

Catalog Number: Price:

Liquid adhesive, 1	NHLG	Call
--------------------	------	------



Durapore™ Tape

Tape a window frame over the polyester net to secure it.

Catalog Number: Price:

1 inch wide x 10 yards; 1 roll	Tape	Call
-----------------------------------	------	------

Maggot Dressings, à la carte



Transparent membrane dressings

Used to secure the maggot dressing to the skin, especially for wounds over areas prone to excessive movement (i.e., joints), perspiration or soiling (i.e., sacrum, coccyx). Placed as a frame over the net-hydrocolloid bond and extending onto the peripheral skin, can make maggot dressings more durable in areas predisposed to soiling and loosening (i.e., wounds over sacrum, coccyx, joints).

	<u>Catalog Number:</u>	<u>Price:</u>
2 inches x 3 inches (brand may vary)	Tegaderm	Call



Skin protective wipe

Optionally used to protect the healthy skin around the wound from rash, maceration or damage from the adhesives and wound drainage (assorted brands).

	<u>Catalog Number:</u>	<u>Price:</u>
Skin protective wipe (brand may vary)	SknPrep	Call

Maggot Therapy for Horses and Other Animals

Not just for humans, Medicinal Maggots™ have proven to be a highly effective treatment for infected and non-healing animal wounds too. Our products have been widely utilized in veterinary care providing a natural and inexpensive solution for hoof and other wounds. Contact us at 949-679-3000 or sales@monarchlabs.com to order and request special pricing. We also manufacture custom veterinary products upon request and with advanced notice. Additionally, we can arrange to have an equine maggot therapy consultant available to answer your questions by phone or video, if necessary.



Disinfection processing under sterile hood

Monarch Labs' insectary--where it all begins!



Our highest priority is making safe and effective wound therapies. Since 2005, we have produced Medicinal Maggots™ (disinfected *Phaenicia sericata* larvae) in our Irvine, California, facility adjacent to John Wayne/Santa Ana Airport. We ship throughout the continental United States via FedEx overnight courier.

Medicinal Maggots™ are a natural, living, wound treatment with an extensive history of use dating back to the American Civil War. Infected horse hoofs can also be economically treated with maggot therapy.

Medicinal Maggots™ are regulated by the Food and Drug Administration as a *prescription only* medical device for treating wounds including pressure ulcers, venous stasis ulcers, neuropathic (diabetic) foot ulcers and non-healing traumatic or post-surgical wounds.

As a living organism, Medicinal Maggots™ are perishable and require special shipping and dressings, which we also sell. Applying maggot therapy is a painless, quick, and easy process. Our specially constructed dressings facilitate a seamless 3-step procedure.

Medicinal Maggots™ are reimbursed by major insurance companies. Over 3,000 health care institutions and veterinarians order our products. They are the first living animal to receive marketing clearance by the Food and Drug Administration!

The Monarch team values innovation, quality control, and exceptional customer service. Call or email us with orders, questions or comments. We are glad to assist--particularly if you are new to maggot therapy.